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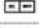
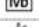
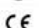
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
ODKAZY

- Goldstein DE, Little RR, Lorenz RA, Malone JI, Nathan D, Peterson CM. Tests of glycemia in diabetes. *Diabetes Care* 1995; 18:896-909.
- Bunn HF. Nonenzymatic glycosylation of protein: relevance to diabetes. *Am J Med* 1981; 70:325-30.
- Jovanovic L, Peterson CM. The clinical utility of glycosylated hemoglobin. *Am J Med* 1981; 70:331-8.
- Nathan DM, Singer DE, Hurxthal K, Goodson JD. The clinical information value of the glycosylated hemoglobin assay. *N Engl J Med* 1984; 310:341-6.
- Goldstein DE, Little RR, Wiedmeyer HM, England JD, McKenzie EM. Glycated hemoglobin: methodologies and clinical applications. *Clin Chem* 1986; 32:B64-70.
- Goldstein DE, Little RR, England JD, Wiedemeyer H-M, McKenzie E. Methods of glycosylated hemoglobins: high performance liquid chromatography and thiobarbituric acid colorimetric methods. In: Clarke WL, Larner J, Pohl SL, eds. *Methods in diabetes research*, Vol. 2. New York: John Wiley, 1986:475-504.
- Tahara Y, Shima K. The response of GHb to stepwise plasma glucose change over time in diabetic patients. *Diabetes Care* 1993; 16:1313-4.
- Svensden PA, Lauritzen T, Soegaard U, Nerup J. Glycosylated haemoglobin and steady-state mean blood glucose concentration in type 1 (insulin-dependent) diabetes. *Diabetologia* 1982; 23:403-5.
- Cefalu WT, Wang ZQ, Bell-Farrow A, Kiger FD, Izlar C. Glycohemoglobin measured by automated affinity HPLC correlates with both short-term and long-term antecedent glycemia. *Clin Chem* 1994; 40:1317-21.
- Singer DE, Coley CM, Samet JH, Nathan DM. Tests of glycemia in diabetes mellitus. Their use in establishing a diagnosis and in treatment. *Ann Intern Med* 1989; 110:125-37.
- Molnar GD. Clinical evaluation of metabolic control in diabetes. *Diabetes* 1978; 27:216-25.
- UK Prospective Diabetes Study. Reduction in HbA1c with basal insulin supplement, sulfonylurea or biguanide therapy in maturity-onset diabetes. *Diabetes* 1985; 34:793-8.
- Baker JR, Johnson RN, Scott DJ. Serum fructosamine concentrations in patients with type II (non-insulin-dependent) diabetes mellitus during changes in management. *BMJ (Clin Res Ed)* 1984; 288:1484-6.
- Tahara Y, Shima K. Kinetics of HbA1c, glycated albumin, and fructosamine and analysis of their weight functions against preceding plasma glucose level. *Diabetes Care* 1995; 18:440-7.
- Brooks DE, Devine DV, Harris PC, et al. RAMP(TM): A rapid, quantitative whole blood immunochromatographic platform for point of care testing. *Clin Chem* 1999; 45:1676-1678.

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